


D • R Burton Healthcare OxyJet® System

Instructions for Use

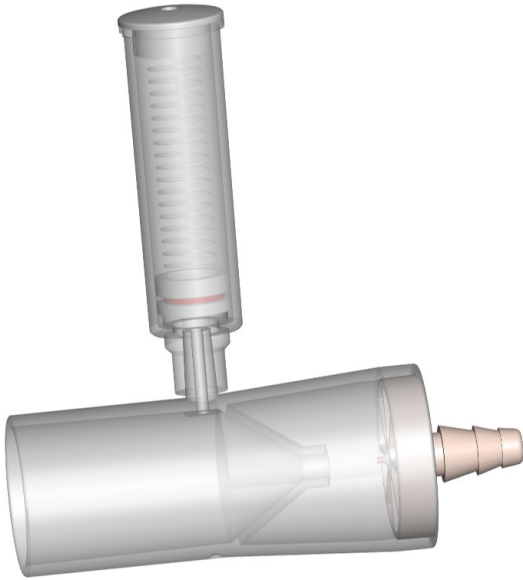


R_x Only




Using the OxyJet with PocketPEP®

1. Attach the Jet T adapter to the PocketPEP by inserting the PocketPEP mouthpiece into the 28mm oval port on the Jet T adapter
2. Attach the OxyJet to the 22mm circular inlet of the Jet T adapter
3. Attach the included manometer to the supplemental port on the Jet T adapter
4. Attach one end of the supplemental connecting tubing to a flowmeter (21% or greater) and the other end to the OxyJet barbed fitting
5. Set the flowmeter to 5 LPM and instruct the patient to take a few breaths
6. Adjust the flowmeter to achieve pressures between 10 - 20 cmH₂O on the manometer while the patient is exhaling into the device. **Do not exceed 15 LPM on the flowmeter.**
7. Clean the OxyJet as needed by following the instructions included with your PocketPEP device

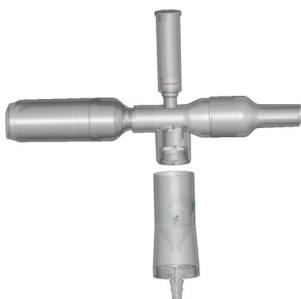

Using the OxyJet with vPEP®

1. Attach the OxyJet to the inlet of the vPEP device by adapting the OxyJet to the rear of the vPEP device
2. Attach the included manometer to the supplemental port on the OxyJet
3. Attach one end of the supplemental connecting tubing to a flowmeter (21% or greater) and the other end to the OxyJet barbed fitting
4. Set the flowmeter to 5 LPM and instruct the patient to take a few breaths
5. Adjust the flowmeter to achieve pressures between 10 - 20 cmH₂O on the manometer while the patient is exhaling into the device. **Do not exceed 15 LPM on the flowmeter.**
6. Clean the OxyJet as needed by following the instructions included with your vPEP device

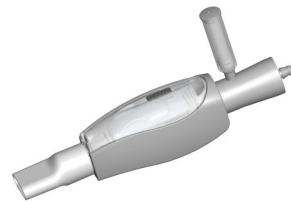

Warnings

- THE OxyJet IS INTENDED FOR USE IN CONJUNCTION WITH THE D • R BURTON HEALTHCARE PocketPEP OR vPEP DEVICES
- DO NOT INCREASE FLOWMETER ABOVE 15 LPM
- IT IS NOT RECOMMENDED TO ALLOW PRESSURE ABOVE 20 cmH₂O DURING BREATHING
- THE OxyJet IS NOT INTENDED FOR USE WITH INVASIVE VENTILATION

Single patient, multi-use

Do not share or use on multiple patients

Not made with natural rubber latex



D • R Burton Healthcare LLC
3936 South Fields Street
Farmville, NC 27828

Telephone: (252) 228 - 7038

Web: www.drburtonhpi.com

Reorder # 561

OxyJet, vPEP, PocketPEP, and D R Burton are registered trademarks of D • R Burton Healthcare LLC.

©2020 D • R Burton Healthcare LLC. All rights reserved. Patent 9,233,274 and other patents pending.